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REFERENCE	
KMINA - PADDED BATHTUB GRAB RAIL (GREY)	K30023
KMINA - PADDED BATHTUB GRAB RAIL (BLUE)	K30024

The **KMINA PADDED BATHTUB GRAB RAILS** are ideal for people with difficulty getting in, risk of falling or reduced mobility, providing assistance and safety for accessing and getting out of the bath, as well as a point of support during use. Do not use this product without reading and understanding the instructions. If necessary, contact a health care professional or technical personnel. Material: white lacquered steel.

#### ASSEMBLY OF THE GRAB RAIL (Fig. 1)

1. Insert the elongated part of the bracket of adjustment (1) into the access bar (2) and use the screwdriver (3) to tighten the three parts so that they fit as shown in Figure 1. The access bar has a non-slip surface on the inside to prevent scratches on the bathtub.

#### INSTALLING THE GRAB RAIL IN THE BATHTUB (Fig. 2)

- Determine the installation location on the rim of the bath.
- Tighten the screwdriver as tight as possible so that it is held securely in place.
- Pull the bar firmly to confirm that it is secure and ready for use.

#### CARE AND CLEANING

- Clean with a microfibre cloth with warm, soapy water.
- For stubborn stains, you can use a sponge, but it is essential not to use scouring pads, bleach or abrasive products.
- We recommend cleaning before first use, after each use and whenever there is contact with body fluids.

#### GENERAL WARNINGS

- For correct use of the device, always consult your doctor or therapist.
- Ensure that the product is delivered complete and keep it packaged away from heat sources. Do not use the product if it is damaged.
- The support bar supports 250 lb.
- Use by one user at a time is recommended.
- Check the adjustment each time you use the access bar.
- Use the device for the purpose for which it was designed. The company is in no way responsible for any misuse of the bathtub access bar, or for injury or damage resulting from such misuse.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

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